




**EZTEST® BIOLOGICAL TEST PACK  
WITH  
SELF-CONTAINED BIOLOGICAL INDICATORS**

**7953<sup>(1)</sup> *G. stearothermophilus***

- EZTest Biological Test Pack contains an EZTest self-contained biological indicator.
- Test pack is small and easy to use, approximately the size of a deck of playing cards.
- Parallels performance of AAMI Test Pack. Record card enclosed in each pack with chemical indicator.
- EZTest self-contained biological indicator contains a known quantity of bacterial spores inoculated onto filter paper and placed inside a plastic culture tube with a crushable glass ampoule containing the culture medium.
- Available for steam processes and contains the spores of *Geobacillus stearothermophilus*. A chemical indicator on the label changes color from blue to black when sterilization exposure has occurred.
- EZTest Biological Test Pack approved by FDA.

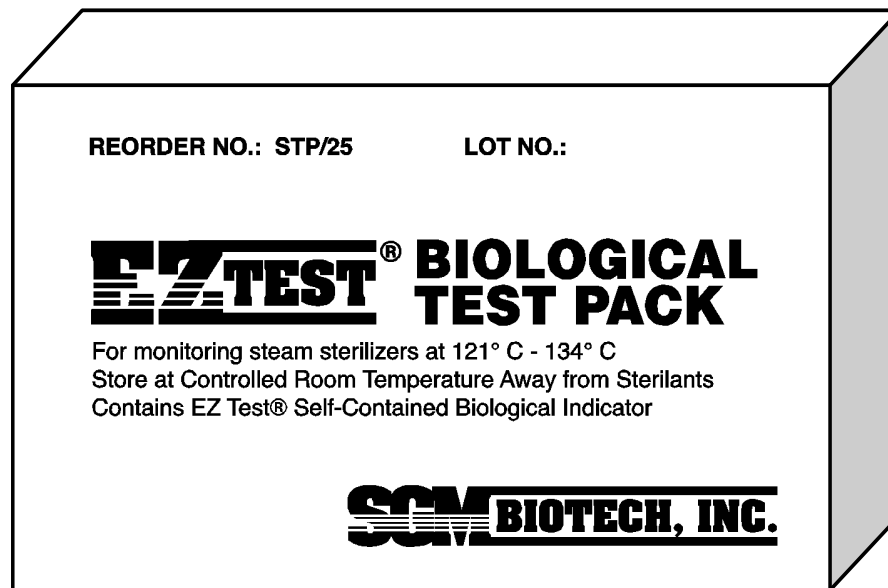

Actual Size  
Dimensions: 4 1/2" x 2 5/8" x 3/4"

<sup>(1)</sup> Culture is traceable to a recognized culture collection identified in USP and ISO 11138.

**Instructions for Use:**

1. Load sterilizer.
2. Lay EZTest biological test pack flat on lowest shelf.
3. Process load with a normal cycle.
4. Remove the test pack – allow to cool at room temperature for 15 minutes.
5. Remove the EZTest biological indicator and incubate at 55-60°C for 24 hours.

**EZTest Advantages:**

- Convenient to culture.
- Results are final in 24 hours.
- Dramatic color change for positive cultures.
- No special laboratory is necessary for culturing.

**Certification:**

EZTest biological indicators are certified for population, D-value, Z-value and performance. EZTest biological indicators are manufactured in compliance to USP, ISO 11138 and ISO 11140-1.

**EZTest Biological Test Packs are available as follows:**

Units per case    Cat No.

	<b>Units per box</b>	<b># of Controls</b>	<b>Reorder Number</b>
<i>G. stearothermophilus</i> 10 <sup>5</sup>	25	5	STP/05
<i>G. stearothermophilus</i> 10 <sup>5</sup>	100	25	STP/25

**Storage:**        Room Temperature. Do Not Desiccate.

**Expiration:**    24 months from date of manufacture

Rev. 4  
Date: 11/1/06  
SGM Part #: 7807