



Smart-Read™ EZTest® BIOLOGICAL TEST PACK
SELF-CONTAINED BIOLOGICAL INDICATORS

7953⁽¹⁾ *G. stearothermophilus*

- Smart-Read EZTest Biological Test Pack contains a Smart-Read EZTest self-contained biological indicator.
- Test pack is small and easy to use, approximately the size of a deck of playing cards.
- Parallels performance of AAMI Test Pack. Record card enclosed in each pack with chemical indicator.
- Smart-Read EZTest self-contained biological indicator contains a known quantity of bacterial spores inoculated onto filter paper and placed inside a plastic culture tube with a crushable glass ampoule containing the culture medium.
- Available for steam processes and contains the spores of *Geobacillus stearothermophilus*. A chemical indicator on the label changes color from blue to black when sterilization exposure has occurred.
- Smart-Read EZTest Biological Test Pack approved by FDA.



Actual Size

Dimensions: 4 1/2" x 2 5/8" x 3/4"

⁽¹⁾ Culture is traceable to a recognized culture collection identified in USP and ISO 11138.

Instructions for Use:

1. Load sterilizer.
2. Lay Smart-Read EZTest biological test pack flat on lowest shelf.
3. Process load with a normal cycle.
4. Remove the test pack – allow to cool at room temperature for 15 minutes.
5. Remove the Smart-Read EZTest biological indicator and incubate at 57-62°C for 10 hours.

Smart-Read EZTest Advantages:

- Convenient to culture.
- Results are final in 10 hours.
- Dramatic color change for positive cultures.
- No special laboratory is necessary for culturing.

Certification:

Smart-Read EZTest biological indicators are certified for population, D-value, Z-value and performance. Smart-Read EZTest biological indicators are manufactured in compliance to USP, ISO 11138 and ISO 11140-1.

Smart-Read EZTest Biological Test Packs are available as follows:

	Units per case	# of Controls	Reorder Number
<i>G. stearothermophilus</i> 10 ⁵	25	5	SRSP/05
<i>G. stearothermophilus</i> 10 ⁵	25	25	SRSP/25
<i>G. stearothermophilus</i> 10 ⁶	25	5	SRSP6/05
<i>G. stearothermophilus</i> 10 ⁶	25	25	SRSP6/25

Storage: Room Temperature. Do Not Desiccate.

Expiration: 24 months from date of manufacture